

Diabetes insipidus after pituitary surgery

Direct testing with Copeptin proAVP

Pituitary surgery
(transsphenoidal or cranial)

Low Copeptin proAVP
(≤12h post-surgery level vs. baseline)


Diabetes insipidus

Low 12h post-operative Copeptin proAVP levels despite surgery-induced stress indicate later Diabetes insipidus. A lack of an increase of Copeptin proAVP within twelve hours after surgery indicates a **deficient function of the posterior lobe of the pituitary** and is therefore a reliable marker for the development of post-operative Diabetes insipidus.¹

High Copeptin proAVP
(≤12h post-surgery level vs. baseline)

Uneventful course

A high 12h-post-operative Copeptin proAVP level is strongly predictive of an uneventful post-operative course in terms of Diabetes insipidus.¹ In patients with an **intact posterior pituitary lobe function**, Copeptin proAVP levels increase due to surgery-induced stress within the first 12 hours after surgery.

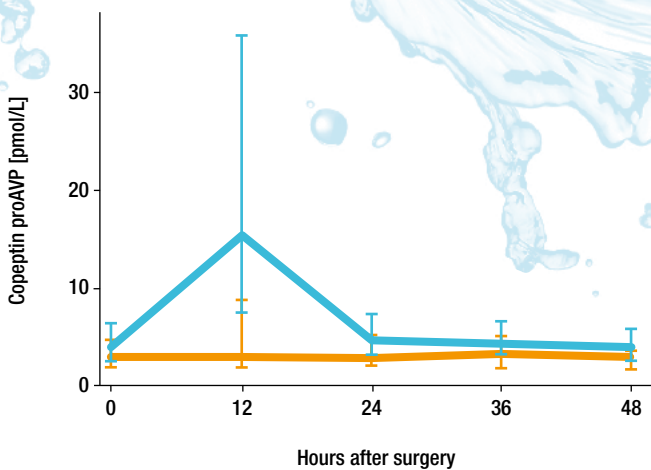

	Copeptin proAVP values	
	Pre-operative (pM, median [IQR])	Post-operative (pM, median [IQR])
Uneventful course (n=155)	3.92 [2.5-6.5]	10.8 [5.2-30.4]
Diabetes insipidus (n=50)	2.9 [1.9-4.7]	2.9 [1.9-7.9]
	p=0.04	p<0.001

Figure 1 Copeptin proAVP measurements at different time points in patients before (0 hours) and after pituitary surgery.¹

Copeptin proAVP – a promising novel tool in the early goal directed management of patients after pituitary surgery

Surgery is a stressful event known to stimulate hypothalamic stress hormone release.^{2,3} Maximal stress is generally experienced after extubation and it has been shown that Copeptin proAVP levels multiplied after surgical treatment (Figure 2).⁴

Manipulation of the pituitary gland during neurosurgery may alter its secretory function. **16-34% of patients undergoing pituitary surgery developed a post-operative Diabetes insipidus (DI).**⁵ DI is therefore the most common cause of prolonged hospital stay.⁶

Although the disease is self-limiting and benign in the majority of cases, Diabetes insipidus occasionally develops into severe hypernatremia and hyperosmolality if the deficit of fluids is not immediately replaced.

Therefore, a timely and accurate diagnosis followed by an appropriate patient management is crucial.^{7,8}

Copeptin proAVP, also called Copeptin, forms the C-terminal part of pre-vasopressin. Upon stimulation, vasopressin and Copeptin proAVP are released from their storage granules in the pituitary and rapidly enter the bloodstream in equimolar amounts.^{9,10} **Thus, Copeptin proAVP can be considered a true surrogate marker of vasopressin.**

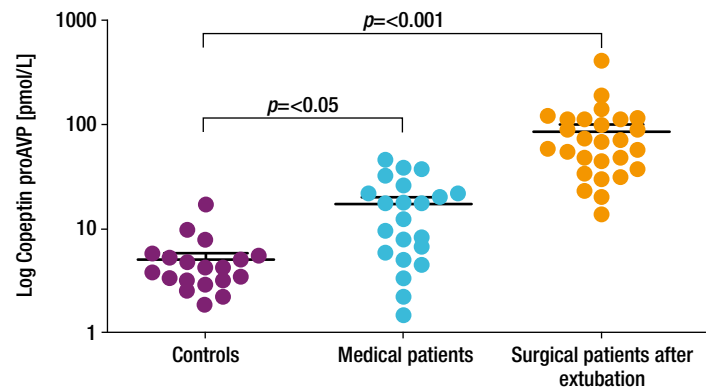


Figure 2 Copeptin proAVP (Copeptin) levels in controls, medical patients and surgical patients after extubation, mirroring three different levels of physical stress⁴

Ask your lab for
Copeptin proAVP

Fast, precise, and smart Copeptin proAVP – the better vasopressin

Your options to replace vasopressin (AVP) assays
with easy and precise Copeptin proAVP (CT-proAVP) assays

- **B-R-A-H-M-S™ Copeptin proAVP KRYPTOR™** (Automated immunofluorescent assay) Article number 857.050
- **B-R-A-H-M-S CT-proAVP LIA** (Immunochemiluminometric assay) Article number 119.100



References

1. Winzler B et al., J Clin Endocrinol Metab 2015; <http://dx.doi.org/10.1210/jc.2014-4527>
2. Donald RA et al., Clin Endocrinol (Oxf) 1993; 38, 609-15
3. Udelsman R et al., J Clin Endocrinol Metab 1987; 64, 986-94
4. Katan M, et al., Neuro Endocrinol Lett 2008; 29, 341-6
5. Hensen J et al., Clin Endocrinol (Oxf) 1999; 50, 431-9
6. Verbalis JG, Pituitary 2002; 5, 119-32
7. Anderson RJ, Kidney Int 1986; 29, 1237-47
8. Berl T, Kidney Int 1990; 37, 1006-18
9. Holwerda DA, Eur J Biochem 1972; 28(3): 334-9
10. Holwerda DA, Eur J Biochem 1972; 28(3): 340-6

www.copeptin.com

© 2015 Thermo Fisher Scientific Inc. All rights reserved. KRYPTOR is a registered trademark of CIS bio international, licensed for use by B-R-A-H-M-S, a part of Thermo Fisher Scientific. All other trademarks are the property of Thermo Fisher Scientific and its subsidiaries. All data regarding specifications, terms and pricing correspond to the existing knowledge at the time of the printing. We are not responsible for any errors, misprints or changes. Reprint, also in parts, solely with prior written consent of B-R-A-H-M-S GmbH.

Thermo Fisher Scientific products are distributed worldwide; not all intended uses and applications mentioned in this printing are registered in every country.

The manufacture and/or use of this product is covered by one or more of the following patents: EP1738178, IN224950, MX262195, US7807397, US12447105, CN200580021434, JP5320294, JP4932714, JP5340160, EP2089718, US8158368, CN101600967, CN2007800452789, HK101016419, CN2011103593804

Clinical Diagnostics

Thermo Fisher Scientific
B-R-A-H-M-S GmbH
Neuendorfstr. 25
16761 Hennigsdorf
Germany

+49 (0)3302 883 0
+49 (0)3302 883 100 fax
info.endo@thermofisher.com

www.thermoscientific.com/brahms
www.thermoscientific.com/copeptin
www.thermoscientific.com/kryptor

Thermo
SCIENTIFIC

A Thermo Fisher Scientific Brand

Learn more on
Copeptin proAVP



copeptin.com