

$$LR = \left( \frac{SD_C}{SD_D} \right) \cdot e^{-0.5 \left[ \left( \frac{\log_{10} X - MD}{SD_D} \right)^2 - \left( \frac{\log_{10} X}{SD_D} \right)^2 \right]}$$

## Powerful and robust

B·R·A·H·M·S Fast Screen pre I plus:  
Risk calculation for prenatal screening

# B·R·A·H·M·S Fast Screen pre I plus

A modern, powerful, and robust clinical  
PNS software for 1<sup>st</sup> and 2<sup>nd</sup> trimester

High quality trisomy and  
pre-eclampsia **algorithms**



**Database** of more than  
**220 000 pregnancies**

**Reliable and effective risk calculation**


Thermo Scientific™ B·R·A·H·M·S™ Fast Screen pre I plus is a **CE marked** software designed to ensure **convenience of data entry, risk calculation and reporting** for laboratories with both, low and high data throughput.



## Because quality matters

First trimester combined screening considers maternal characteristics, biomarkers such as PAPP-A and Free  $\beta$ hCG as well as ultrasound measurements such as Nuchal Translucency (NT) and Nasal Bone (NB).

The result of this risk assessment is provided as individual risk of a pregnant woman. It is essential, that the result is as reliable and accurate as possible. This can only be achieved by using the best available methods.

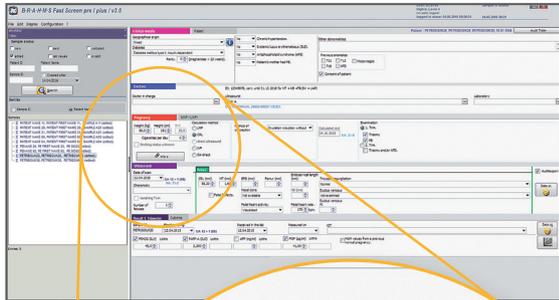

**Pregnancy** MAP / UAPI

Weight (kg) 59,0 Height (cm) 151 BMI 25,9

Cigarettes per day 1

Smoking status unknown

Calculation method

- LMP
- CRL
- direct ultrasound
- IVF
- GA direct

**Ultrasound**

Date of scan 12.04.2015 GA 12 + 1 (85) MA: 31-2

Chorionicity


Vanishing Twin

**Fetus**

CRL (mm) 55,30 NT (mm) 1,60

Fetal Defects.

**Convenient data entry:  
All patient details on  
one screen**

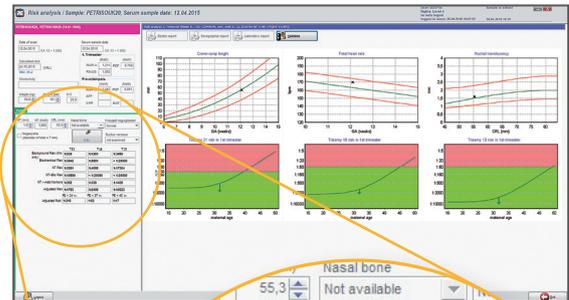

Nasal bone 55,3 Not available

of bleb  $\geq 7$  mm

Info Ductus venosus not examined

	T21	T18	T13
Background Risk (MA only)	1:539	1:1261	1:3969
Biochemical Risk	1:3642	1:8981	< 1:20000
NT Risk	1:2891	1:4089	1:17304
NT+Bio Risk	1:19590	< 1:20000	< 1:20000
NT + Add. Markers	1:262	1:339	1:1435
Adjusted Risk	1:1763	1:2406	1:10323
PE < 34 w.		PE < 37 w.	PE < 40 w.
Adjusted Risk	1:215	1:52	1:17

**Support of decisions:  
Detailed presentation  
of risks**



## Ultrasound

A correct determination of NT is very important for a correct risk calculation. It is recommended that a sonographer holds a certification of the Fetal Medicine Foundation (FMF) <sup>1</sup> or a corresponding local organization.



## Biomarkers

The biochemical assays Thermo Scientific B-R-A-H-M-S Free  $\beta$ hCG, PAPP-A and PIGF plus KRYPTOR™ fulfil the strict quality requirements of the FMF since 1999 and provide constantly the highest precision and stability data as proved by the UK NEQAS data.<sup>2</sup>



## Software

B-R-A-H-M-S Fast Screen pre I plus offers first trimester algorithms that are based on the FMF data and are updated with recently published further improvements. The huge database ensures a statistically significant robustness.<sup>3</sup>

# High quality algorithms and stable medians ensure best results

1<sup>st</sup>

trimester  
algorithms

## Trisomy 21, 18 and 13

The B·R·A·H·M·S Fast Screen pre I plus algorithm for first trimester combined screening is based on FMF risk calculation and published data.<sup>3,4,5</sup>

PAPP-A  
Free  $\beta$ hCG  
PIGF  
NT  
NB  
DVPI  
TR

Week of gestation ... 3 4 5 6 7 8 9 10 11 12 13 14


## Pre-eclampsia

The B·R·A·H·M·S Fast Screen pre I plus algorithm for first trimester pre-eclampsia combines maternal characteristics, obstetrics history, biophysical markers and maternal serum biomarkers.<sup>6</sup> The algorithm developed by the FMF was fitted on gestational age at the time of delivery with pre-eclampsia.<sup>7</sup>

PAPP-A  
PIGF  
MAP  
UAPI

### Stable and robust medians guarantee a reliable risk calculation

The medians of the B·R·A·H·M·S biochemical markers Free  $\beta$ hCG, PAPP-A and PIGF<sup>8</sup>, which are included in the B·R·A·H·M·S Fast Screen pre I plus software, are extremely stable. The medians were established on over 220 000 healthy pregnancies.<sup>3</sup> A recalculation of medians and MoMs over the time is not required.<sup>9</sup>

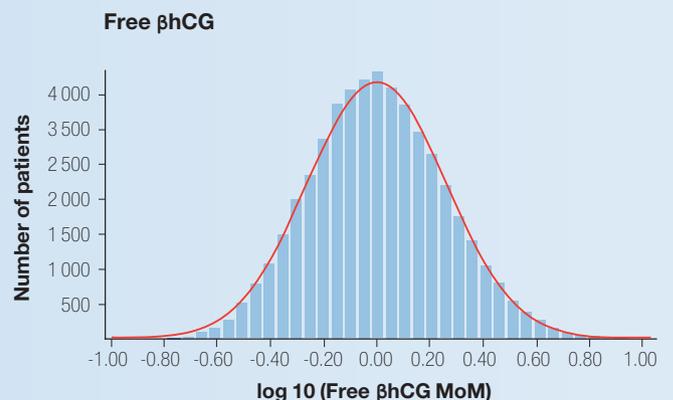

# Trisomy 21 and 18

The B·R·A·H·M·S Fast Screen pre I plus second trimester algorithm allows calculation of risk for trisomies 21 and 18 with double test (AFP and hCG+β/Free βhCG).

**AFP**  
**hCG+β / Free βhCG**

**2<sup>nd</sup>**  
**trimester**  
**algorithms**


15 16 17 18 19 20 21 22 23 24 25 ...

# NTD

The AFP value indicates the fetal neural tube defect (NTD).

**AFP**

- DVPI** Ductus Venosus Pulsatility Index
- MAP** Mean Arterial Pressure
- NB** Nasal Bone
- NT** Nuchal Translucency
- NTD** Neural Tube Defects
- TR** Tricuspid Regurgitation
- UAPI** Uterine Artery Pulsatility Index

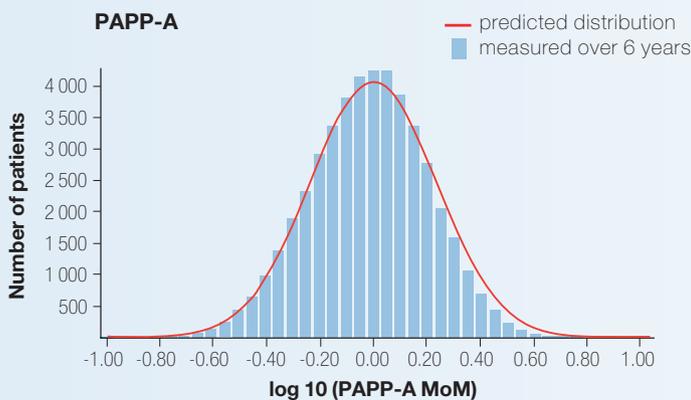

**Figure** Distribution of the cumulative measurement of Free βhCG and PAPP-A of ~55 000 samples (blue bars) over a time period of 6 years compared to the predicted distribution (red curve)<sup>9</sup>

# B·R·A·H·M·S Fast Screen pre I plus

## More than just a risk calculation tool

B·R·A·H·M·S Fast Screen pre I plus contains various useful tools for data management, internal and external quality control, audits and statistics. Particular attention is paid to patient data security and traceability.



### **Maximum data security and traceability: Fulfilling the requirements of a clinical software**

- Software audit option for external quality control
- Internal audit trails to trace all actions
- Pop-up warnings if data is missing or out of range
- Validation of data before production of report
- All data are saved automatically
- 4 levels of users for data entry and management




### **Easy and effective: Useful data management tools**

- Statistic and audit reports
- Pregnancy outcome requests and follow-up information
- Database search tool
- Export data for statistics and research
- Printing reports one by one or in defined batches
- Printing, saving and sending reports automatically via email
- Possibility to keep extended patient data for record purposes


### **Additional benefits**

- 7 languages of interface, reports and user manuals (English, German, French, Spanish, Italian, Turkish, and Russian)
- Software trial version for 60 days
- Dedicated technical and scientific support

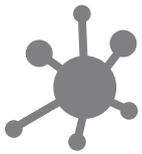


**High connectivity:**  
Meeting the lab's need

## **Manual** entry of patient data and **connection to B·R·A·H·M·S KRYPTOR**

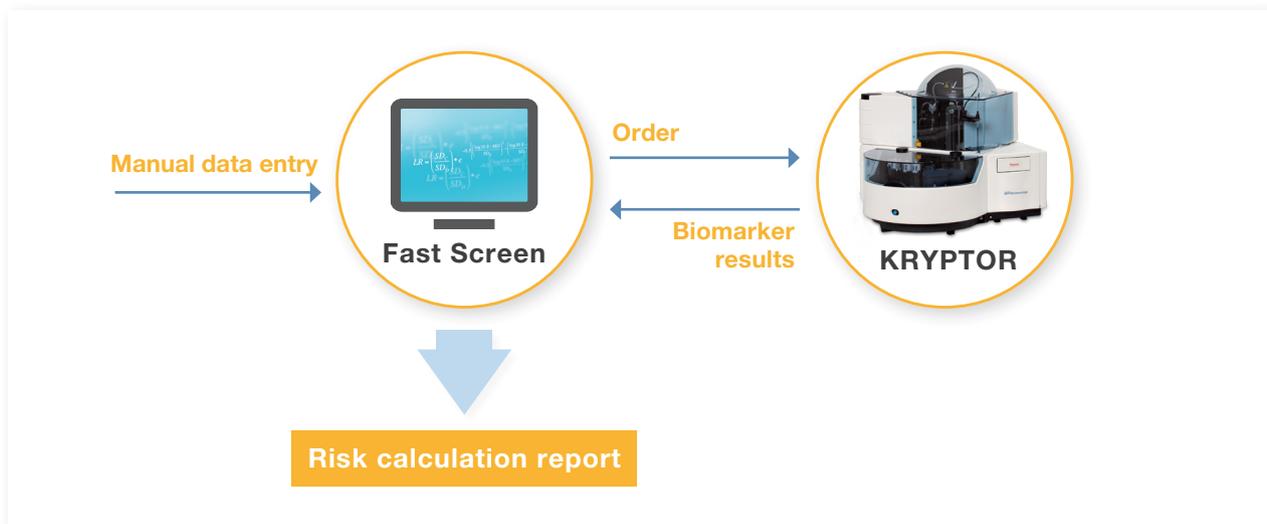

## **Automatic** data transfer via Laboratory Information System (LIS)

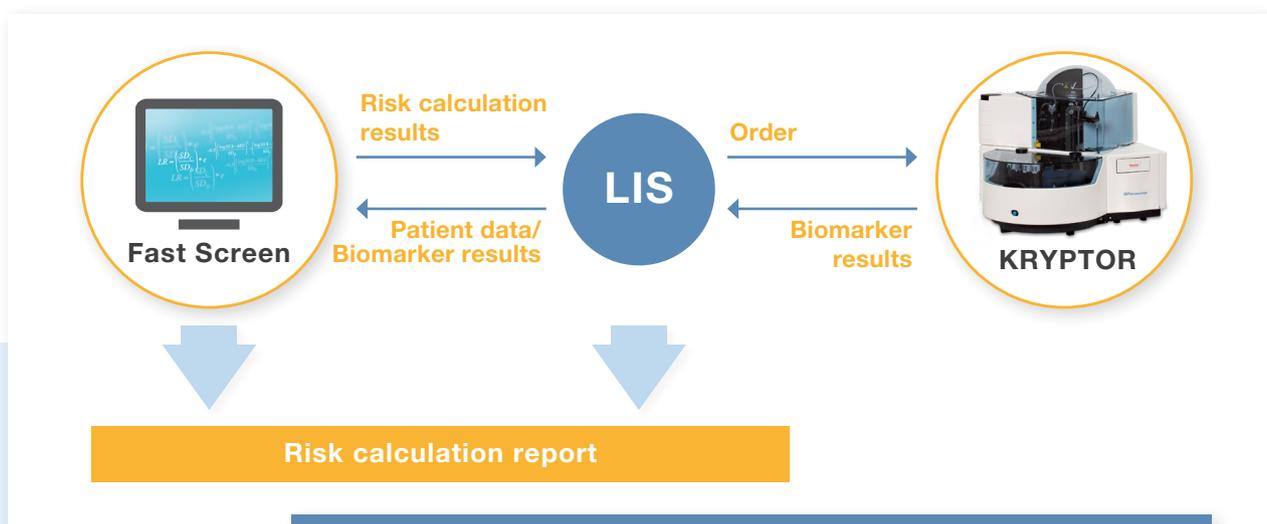


Automated transfer of all data either patient by patient (manual mode) or in defined batches (auto mode)

CE marked  
clinical software



Visit our online  
software demonstration  
[fastscreen.world-of-biomarkers.com](https://fastscreen.world-of-biomarkers.com)

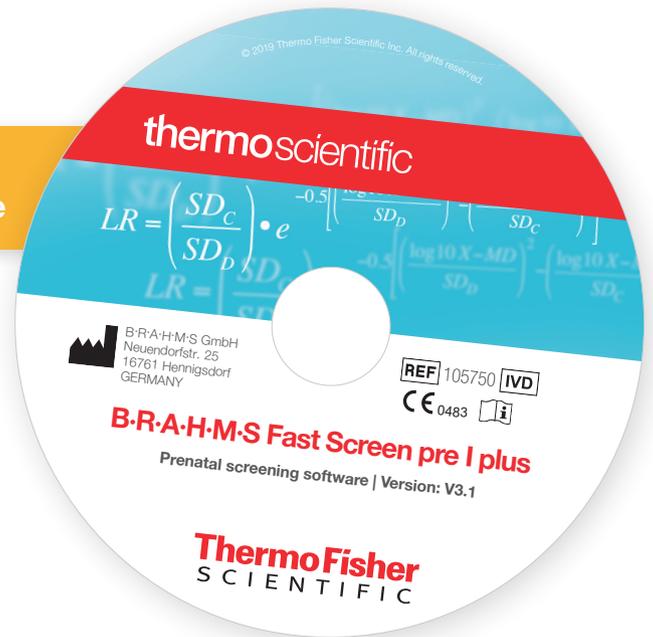

## Thermo Scientific B-R-A-H-M-S Biomarkers Prenatal Screening Portfolio on B-R-A-H-M-S KRYPTOR Systems



### Biomarkers

B-R-A-H-M-S AFP KRYPTOR	Art. no.: 816.075
B-R-A-H-M-S Free $\beta$ hCG KRYPTOR	Art. no.: 809.075
B-R-A-H-M-S hCG+ $\beta$ KRYPTOR	Art. no.: 841.050
B-R-A-H-M-S Inhibin A KRYPTOR*	Art. no.: 850.075
B-R-A-H-M-S PAPP-A KRYPTOR	Art. no.: 866.075
B-R-A-H-M-S PIGF plus KRYPTOR**	Art. no.: 859.075
B-R-A-H-M-S sFlt-1 KRYPTOR**	Art. no.: 845.075
B-R-A-H-M-S uE3 KRYPTOR***	Art. no.: 803.075



### Software

B-R-A-H-M-S Fast Screen pre I plus	Art. no.: 105750
------------------------------------	------------------



### Instruments

B-R-A-H-M-S KRYPTOR GOLD	Art. no.: BMKSYS0201
B-R-A-H-M-S KRYPTOR compact PLUS	Art. no.: 106172

\* Available on B-R-A-H-M-S KRYPTOR GOLD

\*\* Available on B-R-A-H-M-S KRYPTOR compact PLUS and B-R-A-H-M-S KRYPTOR GOLD

\*\*\* Available on B-R-A-H-M-S KRYPTOR, B-R-A-H-M-S KRYPTOR compact PLUS and B-R-A-H-M-S KRYPTOR GOLD

### References

- <https://fetalmedicine.org/training-n-certification/certificates-of-competence>
- Monthly UK NEQAS reports, February 2003 – September 2019
- Wright D et al. Ultrasound Obstet Gynecol 2010; 36: 404-411
- Kagan KO et al. Ultrasound Obstet Gynecol 2008; 31: 618-624
- Nicolaides KH. Prenat Diagn 2011; 31: 7-15
- Wright D et al. Fetal Diagn Ther 2012; 31: 87-93
- O'Gorman N et al. Am J Obstet Gynecol 2016; 214(1): 103.e1-103.e12
- Pandya P et al. Fetal Diagn Ther 2012; 32: 171-178
- Spencer K. Presentation on B-R-A-H-M-S KRYPTOR User Meeting, Cambridge, March 2007

### Clinical Diagnostics

Thermo Fisher Scientific  
B-R-A-H-M-S GmbH  
Neuendorfstr. 25  
16761 Hennigsdorf/Germany

+49 (0)3302 883 0  
+49 (0)3302 883 100 fax  
info.brahms@thermofisher.com  
www.thermoscientific.com/brahms

Find out more at [thermoscientific.com/brahms](https://thermoscientific.com/brahms)

Products are CE marked but not 510(k)-cleared and not available for sale in the U.S. Availability of products in each country depends on local regulatory marketing authorization status.

© 2020 Thermo Fisher Scientific Inc. All rights reserved. All trademarks are the property of Thermo Fisher Scientific and its subsidiaries unless otherwise specified. KRYPTOR is a trademark of Cisbio Bioassays, licensed for use by B-R-A-H-M-S GmbH, a part of Thermo Fisher Scientific.

107486.3